

**INTENDED USE**

Sure/Cut Adson, Ferris-Smith, and DeBakey Tissue Modules are intended for use in surgical closure for cutting suture with **SURE/CUT** Forceps.

DEVICE DESCRIPTION

SURE/CUT Adson Ferris-Smith and DeBakey cutting Modules are removable/replaceable suture cutting modules comprised of a polymer housing containing a stainless-steel cutting blade with included **SURE/CUT** Applicator Guard for easy application and removal.

**INDICATIONS FOR USE**

SURE/CUT Adson, Ferris-Smith and DeBakey Modules are indicated for use to cut various surgical suture sizes from 4-0 to #2 monofilament or multifilament materials.

CONTRAINDICATIONS

None Known

WARNINGS

- DO NOT REUSE disposable **SURE/CUT** cutting modules are single patient use disposable and not intended to be reprocessed or re-sterilized

Sterile packaged disposable **SURE/CUT** cutting module are to be affixed to **SURE/CUT** Forceps prior to use. **NOTE:** Product code AD-100M **SURE/CUT** cutting module, color coded "Purple" to be affixed to **SURE/CUT** Adson Forceps. DB-100M **SURE/CUT** cutting module, color coded "Raspberry" to be affixed to **SURE/CUT** DeBakey Forceps. FS-100M **SURE/CUT** cutting module, color coded "Green" to be affixed to **SURE/CUT** Ferris-Smith Forceps.

- **SURE/CUT** cutting module is not intended for cutting any other materials or any tissue.
- **SURE/CUT** Modules to be attached to forceps prior to the wound closing routine.
- **SURE/CUT** Modules are not interchangeable and not intended for use on any other type forceps.

PRECAUTIONS

- Do not use if the packaging or device is damaged. Review Instructions for Use before using.

INSTRUCTIONS FOR USE**PREPARATION**

1. Dispense **SURE/CUT** Adson Ferris-Smith and DeBakey Modules to sterile field and inspect for damage and that the cutting blade is secure.
2. Grasp **SURE/CUT** Module in one hand by the included "Applicator Guard" and with the **SURE/CUT** Tissue Forceps in the opposite hand. Place the base into the opening at the base of the **SURE/CUT** Module. By applying gentle pressure insert the Tissue Forceps completely into the **SURE/CUT** Module. Remove the Applicator Guard. When procedure is completed, place the Applicator Guard back on the module to remove the module from the Forceps.

USAGE


1. After securely tying sutures, grasp the single strand without the needle for a Running Continuous Closure technique or both strands for an Interrupted Closure Technique. While gently pulling the strand(s) away from the knot at a right angle to the surface of the skin, place the notched blade shields over the strand(s) and slowly slide the **SURE/CUT** Module down until the knot is felt against either one of the Sure/Cut blade shields. Rock the entire forceps up or down to adjust to the desired length of the suture tail(s). Pull the suture tails directly toward the forceps while also gently pushing the forceps and Sure/Cut Module 180 degrees in the opposite direction and towards the suture until the suture separates at the determined length. Inspect the suture tail(s) for proper length and a secure knot. In the event the suture tail(s) is cut too long. Suture tails can be trimmed with scissors.
2. After use, remove the **SURE/CUT** from forceps by grasping the **SURE/CUT** Module Side Grips in one hand and gripping the forceps with the other and pulling them firmly in opposite directions. The Sure/Cut module will release from the forceps with medium strength pull.
3. Dispose of the Sure/Cut device as appropriate in accordance with accepted medical practice and applicable local, state and federal laws and regulations

STORAGE OF THE DEVICE

SURE/CUT Adson, Ferris-Smith and DeBakey Suture Cutting Modules should be stored at normal room temperature conditions within original packaging.

Product Ordering:

To order a device, or for additional product information, please contact:

 **Huck Medical Technologies, Inc.**
111 Cash Street, Suite A
Jacksonville, Tx 75766

Phone: 903-339-1257
Email: customerservice@huckmedtech.com

